

DOWNLOADING FLUXERGY WORKS & SAMPLE TEST IMPORT

Downloading Fluxergy Works

Download link: <https://fluxergy.com/download/Fluxergy-Works-CE.exe>

Registration Key: Ask your Fluxergy Rep for a Registration Key to activate your software

- Laptop Computer minimum requirements (**Laptop is not included**)
 - Operating System
 - Windows 10 v1151
 - Processor: Intel Core i5 2.5GHz or equivalent
 - RAM: 8GB DDR4
 - HDD: 250GB
 - Screen: 1080p
 - USB: 2x2.0 port (for scanner and mouse)
 - Networking: Ethernet port

Installing Fluxergy Works:

Select link and Download Fluxergy Works

Enter the registration key provided to you by Fluxergy.

- a. Note: there is a one-time activation process which **requires an internet connection**. After activation internet access is no longer necessary.

After installing software, create an administrator account.

- a. Note: the administrator account created during Fluxergy Works install is the master administrator and cannot be edited by other administrator accounts.
- b. Note: **if the master administrative account password is lost, it cannot be reset**. To mitigate the risk of losing all administrative access, Fluxergy recommends creating a secondary administrator account immediately after install
- c. If all user admin passwords are lost, users must uninstall and reinstall Fluxergy Works after backup of test files and deletion of test storage folders from previous installs of Fluxergy Works